



Aperio GT 450 DX 1.2 Patch Notice

Leica Biosystems is always looking for ways to further improve our customers' experience using our products. Our software patches often include new features, but also include improvements based on customer input and our own internal testing. Please make sure to read below for the details of this patch so you can get the best out of your product. Thank you for being a customer of Leica Biosystems.

New Features Included in This Patch

Affected component: Controller

This patch release adds support for the optional DICOM feature pack. For details on this update, refer to the document *Aperio GT 450 DX DICOM Upgrade*, MAN-0515.

Open Issues for Release 1.2

Case #	Description
28271	A problem can occur when Aperio SAM DX sends .SVS slide image files in DICOM format to Aperio eSlide Manager if {BARCODEID} is specified as the image filename format on the SAM Configuration page. Some files will not successfully transfer when using this option, while others will. This is considered a low-risk problem as most customers will be using the DICOM feature pack to send DICOM images using the C-Store protocol instead of connecting to Aperio eSlide Manager.

Improvements Included in This Patch

The following improvements were added in this patch.

Case #	Description
28051	An issue was resolved in which the folders that temporarily contained scanned records were not removed once the folder contents were successfully transferred to their final storage location.
28096	An issue that occurred occasionally was resolved in which the GT 450 DX scanner would become unresponsive during scanning.

Aperio GT 450 DX 1.2 Patch Notice
 Part Number/Revision: PN-0040, Revision A
 Date: June 2023

This document applies to Aperio GT 450 DX Controller, Aperio GT 450 DX Console, and Aperio GT 450 DX SAM version 1.2


Copyright Notice

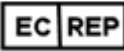
- ▶ Copyright © 2023 Leica Biosystems Imaging, Inc. All Rights Reserved. LEICA and the Leica logo are registered trademarks of Leica Microsystems IR GmbH. Aperio, GT, and GT 450 are trademarks of Leica Biosystems Imaging in the USA and optionally in other countries. Other logos, product and/or company names might be trademarks of their respective owners.
- ▶ This product is protected by registered patents. For a list of patents, contact Leica Biosystems.


User Resources

- ▶ For the latest information on Leica Biosystems Aperio products and services, please visit www.LeicaBiosystems.com/Aperio.

Contact Information – Leica Biosystems Imaging, Inc.

Headquarters	Customer Support
 Leica Biosystems Imaging, Inc. 1360 Park Center Drive Vista, CA 92081 USA Tel: +1 (866)-478-4111 (toll free) Direct International Tel: +1 (760) 539-1100	Contact your local support representative with any query and service request. www.leicabiosystems.com/contact-us/

European Union Authorized Representative	United Kingdom Responsible Person
 CEpartner4U Esdoornlaan 13 3951 DB Maarn The Netherlands	Leica Microsystems (UK) Limited Larch House, Woodlands Business Park Milton Keynes, England, United Kingdom, MK14 6FG

Importers	
 Leica Biosystems Deutschland GmbH Heidelberger Straße 17-19 69226 Nussloch, Germany	Leica Microsystems (UK) Limited Larch House, Woodlands Business Park Milton Keynes, England, United Kingdom, MK14 6FG

Revision Record

Rev	Issued	Sections Affected	Detail
A	June 2023	All	New document.

